
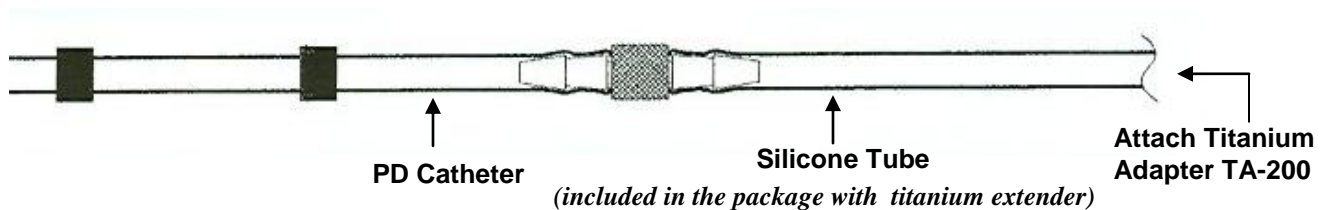


TITANIUM EXTENDER

*for repair or extension of
PD Catheters*

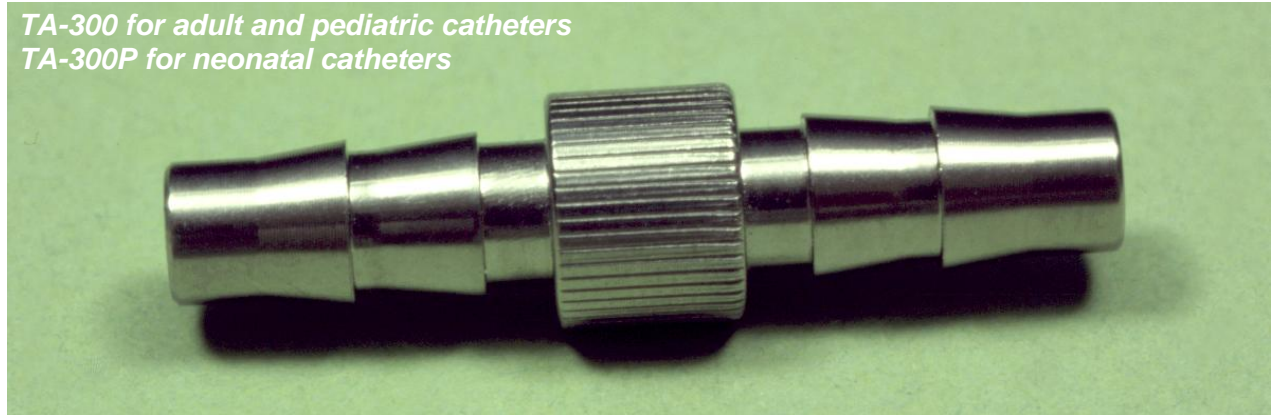

Specifications:

- Catheter Extender is used to repair or extend the external portion of peritoneal catheters
- TA-300 is designed to fit Medionics' 4.95 mm OD PD Catheters
- TA-300P is designed to fit Medionics' Neonatal Catheters
- Manufactured with titanium
- To install, trim the external end of the catheter and slip over one end of the titanium extender
- Slip the supplied piece of silicone tube over the other end of the titanium extender
- Connect titanium adapter to the other end of the silicone tube



Medionics International Inc., 114 Anderson Avenue, Markham, Ontario L6E 1A5 Canada
Tel: (905) 472-6544 Fax: (905) 472-6549 info@medionics.com www.medionics.com

